

Mobilise with Comfort



**NEW:
Version 2.0**

ams active
mobilisation
system

Long-Term Prevention of Pressure Ulcers

The Active Mobilisation System (AMS) mobilises patients continuously and barely perceptibly.



The AMS is an effective support for the prevention and treatment of pressure ulcers with a view to avoiding injury or reducing skin damage.

It provides the care staff with an effective and well accepted offloading alternative particularly for patients that have difficulty with regular turning protocols (such as in the case of dementia, pain, agitation or aggression).


Studies:

- ✓ Schaffhauser et al: Case study: Active Mobilisation System; *Arzt Spital Pflege*; Edition N° 4/2016
- ✓ A. Christen; Active Mobilisation System (AMS) Observational study at the University Clinic for Acute Geriatrics, Stadtspital Waid Zürich (Switzerland) from February to May 2015; published on the compliant concept web site.
- ✓ M. Wettstein; Sunnewies Elderly Home observational study from 2014; published on the compliant concept web site.
- ✓ K. Gläsche-Mehar; Wound and Stoma consultant at the Nottwil Swiss Paraplegic Center: AMS observational study December 2014; published on the compliant concept web site.



Relaxed Thanks to Undisturbed Sleep

- ✓ Gentle turning
- ✓ Relief for patient and staff
- ✓ Better sleep quality
- ✓ More time for care

The AMS can be inserted in any bed easily. Its operation is also quite simple.

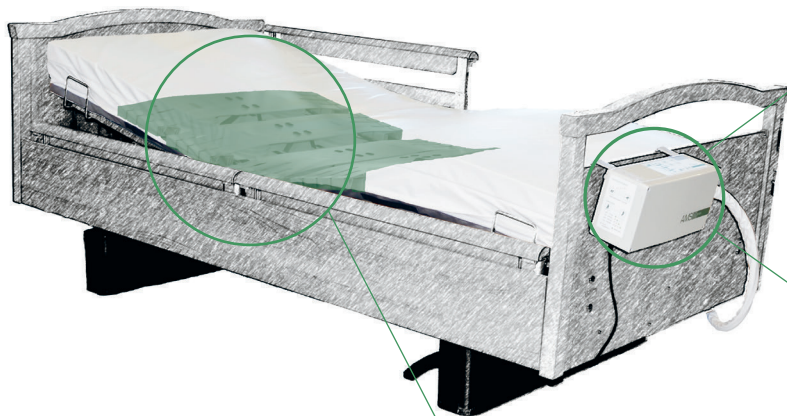

The AMS is virtually noiseless (30dB) and therefore does not affect patient's sleep.

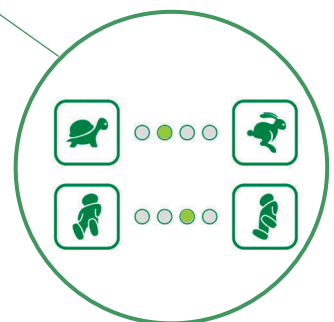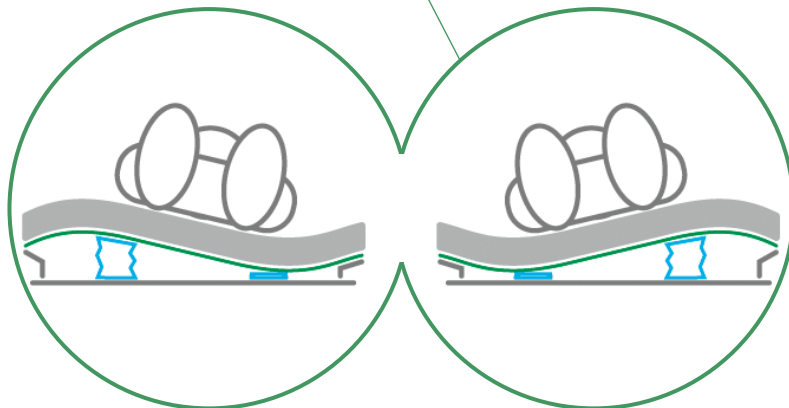


Intervals and intensity of the supports can be easily set on the control unit at the bed.

The active modules smoothly position the person in the bed from one side to the other and thereby continuously shift the pressure points. In addition, pressure on particularly critical parts of the body is relieved.

New (Version 2.0): all programs can run up to 30° back rest incline

Comfort for Continuous Use

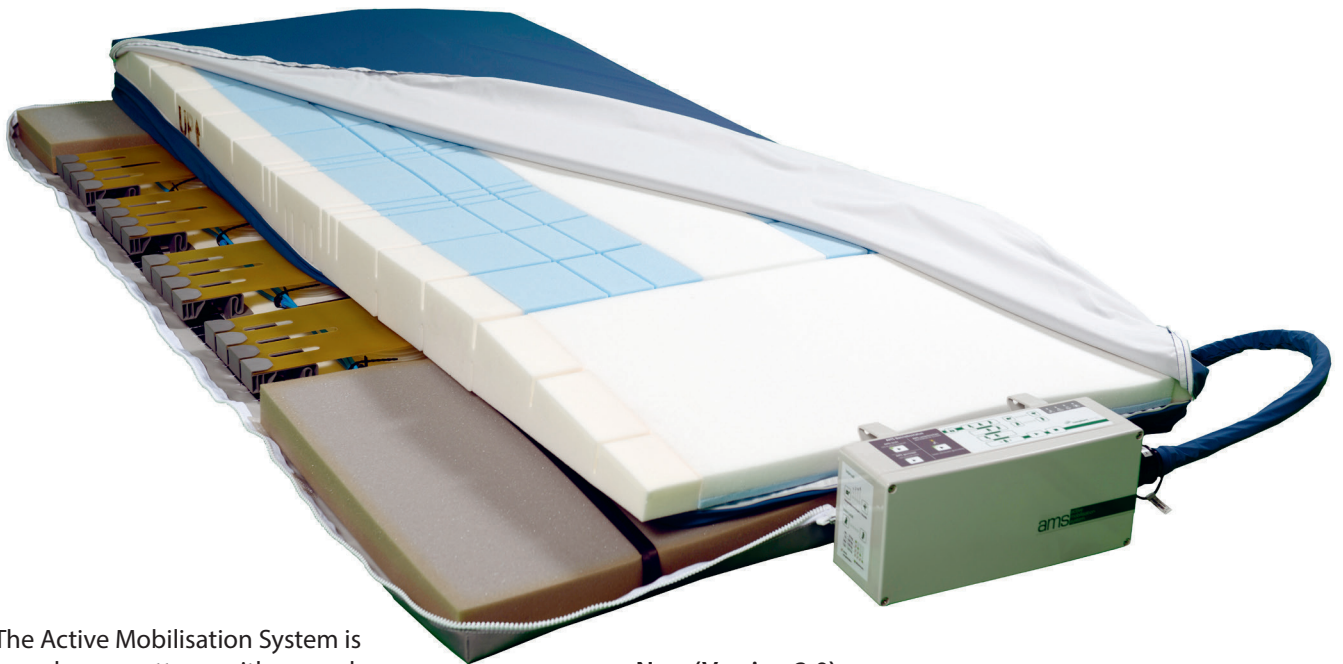

The Active Mobilisation System is based on a mattress with normal lying properties. The system facilitates maintaining the patients' mobility resources and body awareness.

New (Version 2.0):
The heel zone is tapered and has additional viscoelastic foam to provide more protection for heels



Reinforced border zones on the AMS mattress allow safe and comfortable transfers.

Our mutual goal:
Satisfied patients and residents thanks to needs-oriented care

- ✓ Gentle turning
- ✓ Relief for patient and staff
- ✓ Better sleep quality
- ✓ More time for care





compliantconcept

innovations for a better life



compliant concept AG
Undermülistrasse 28
CH-8320 Fehraltorf
Tel: +41 44 552 15 00
Fax: +41 44 552 15 09
info@compliant-concept.ch
www.compliant-concept.ch
DOC-000020.en - 2.0.2